



FUTURE

MOBILITY HEALTHCARE INC.

HEADREST

PARTS MANUAL

FUTURE MOBILITY HEALTHCARE APPRECIATES YOUR BUSINESS.
OUR CUSTOMER SERVICE REPRESENTATIVES CAN BE CONTACTED AT:

**TO PLACE ORDERS:
(905) 671-1661**

FOR YOUR CONVENIENCE YOU MAY ALSO PLACE ORDERS BY FAX AT:

FAX: (905) 671-3377

PLEASE VISIT OUR WEBSITE AT:

www.futuremobility.ca


*Part prices and configurations included in this Parts Manual are subject to change without prior notice.

* All prices are shown as per unit individual price.

Contents

HEADREST VERTICLE TUBES	5
HEADREST FIXED VERTICLE TUBE	5
HEADREST MULTI-AXIAL VERTICLE TUBE	7
HEADREST INFINITE VERTICLE TUBE	9
HEADREST HORIZONTAL TUBES	11
HEADREST FIXED HORIZONTAL TUBE	11
HEADREST ARTICULATED HORIZONTAL TUBE	11
HEADREST INFINITE HORIZONTAL TUBE	11
LINKS HEADREST	13
LINKS HEADREST STRAIGHT VERTICLE TUBE	13
LINKS HEADREST CURVED VERTICLE TUBE	15
LINKS HEADREST BALL JOINT	17
LINKS HEADREST SINGLE LINK	17
HEADREST BASE ADAPTOR BRACKET	19
2-HOLE HEADREST BASE ADAPTOR BRACKET	19
UNIVERSAL HEADREST BASE ADAPTOR BRACKET	21

HEADREST VERTICLE TUBES
HEADREST FIXED VERTICLE TUBE

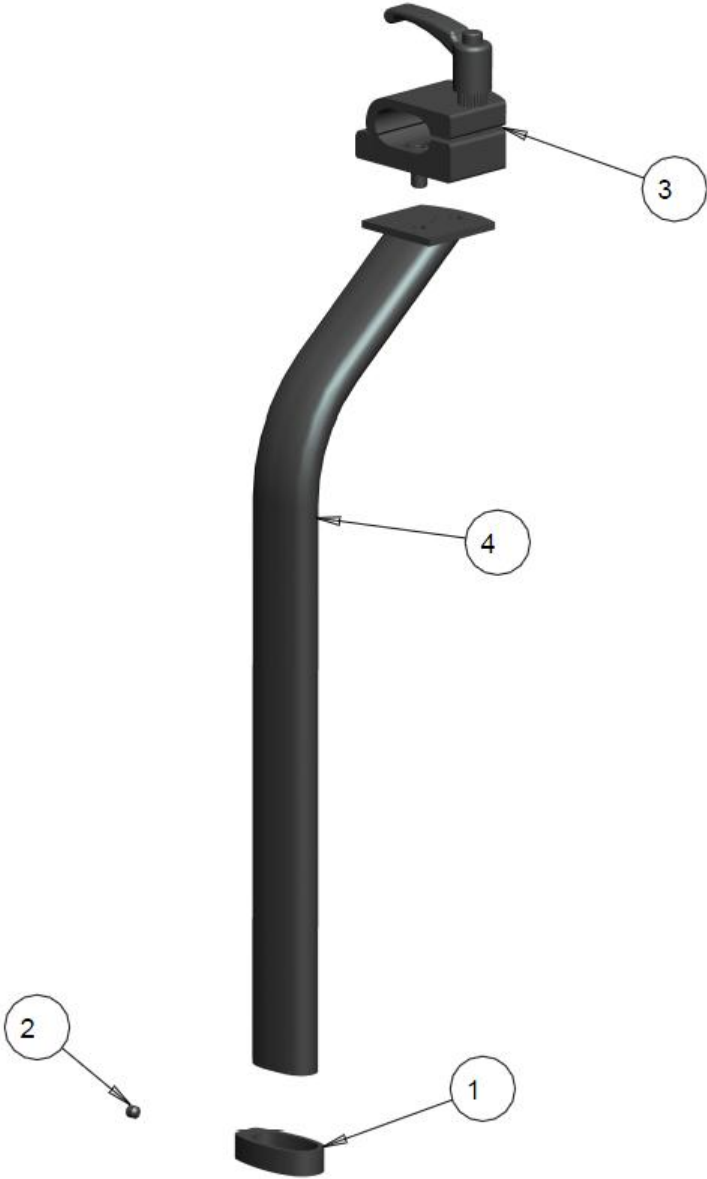


HEADREST FIXED VERTICLE TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	1		1001199	HEADREST LOCKING COLLAR	\$12.00
2	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$2.00
3	1		1019596	CURVED HINGED OVAL HEADREST CLAMP	\$48.00
4	1		1001393	HEADREST CURVED POST WELDMENT	\$52.00
	1		1003928	HEADREST CURVED POST WELDMENT + 4"	\$72.00
	1		1018183	HEADREST STRAIGHT POST WELDMENT	\$47.00
	1		1021424	KIT, CURVED POST STANDARD (ITEMS 1-4)	\$110.00
	1		1021425	KIT, CURVED POST PLUS 4" (ITEMS 1-4)	\$135.00
	1		1021426	KIT, STRAIGHT POST STANDARD (ITEMS 1-4)	\$105.00

NOTES:

HEADREST MULTI-AXIAL VERTICLE TUBE

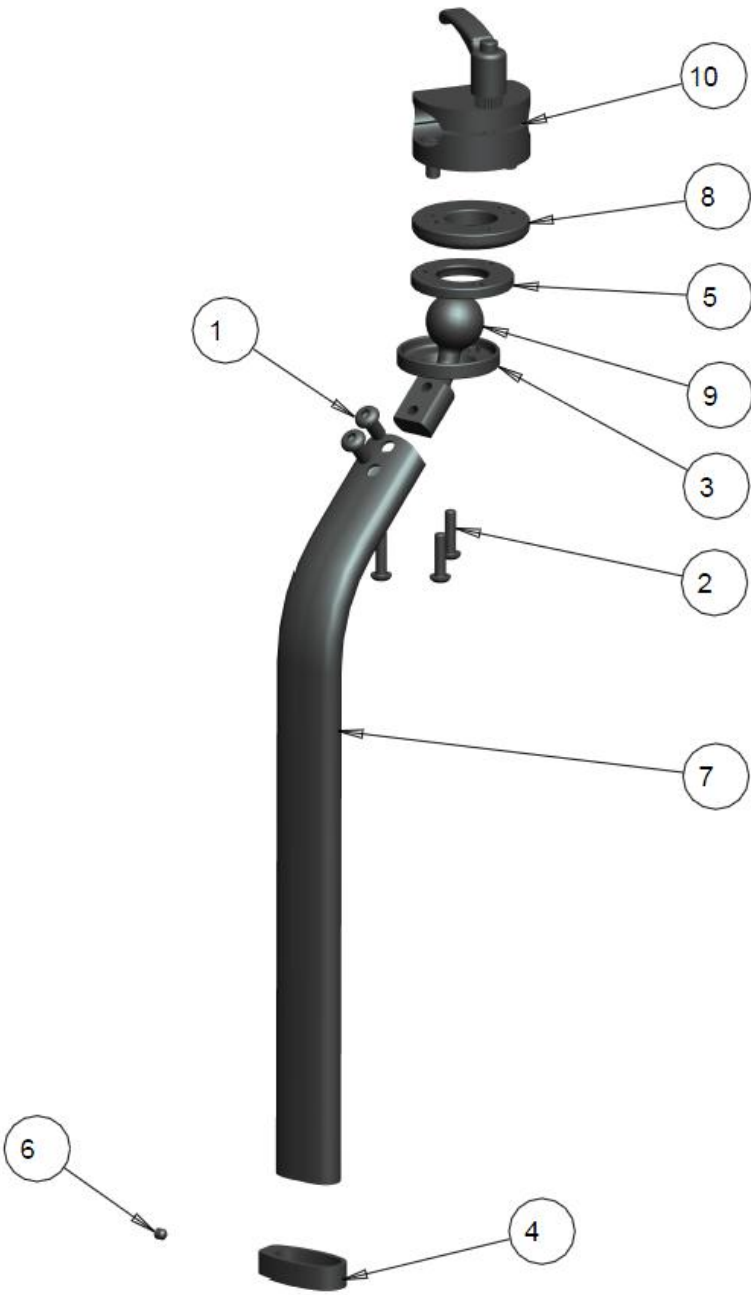

HEADREST MULTI-AXIAL VERTICLE TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	3		1000937	SCREW, SOCKET HEAD BUTTON, #10-32NF X 3/4	\$3.00
3	1		1001198	FACE PLATE, BALL SOCKET CLAMP	\$14.00
4	1		1001199	HEADREST LOCKING COLLAR	\$12.00
5	1		1001200	SPACER PLATE, BALL SOCKET CLAMP	\$12.00
6	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$2.00
7	1		1007004	MULTIAXIAL HEADREST CURVED VERT POST WELDMENT	\$52.00
	1		1007005	MULTIAXIAL HEADREST CURVED VERT POST WELDMENT +4"	\$72.00
	1		1020201	MULTIAXIAL HEADREST STRAIGHT VERT POST WELDMENT	\$47.00
8	1		1005793	MOUNT PLATE, BALL SOCKET CLAMP	\$17.00
9	1		1007001	BALL-JOINT, 1" DIA CURVED	\$27.00
10	1		1020987	CIRCULAR HINGED HEADREST CLAMP	\$58.00
	1		1007007	KIT, MULTI CURVED POST STANDARD (ITEMS 1-10)	\$180.00
	1		1007008	KIT, MULTI CURVED POST +4" (ITEMS 1-10)	\$200.00
	1		1021427	KIT, MULTI STRAIGHT POST STANDARD (ITEMS 1-10)	\$175.00

NOTES:

HEADREST INFINITE VERTICLE TUBE




HEADREST INFINITE VERTICLE TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001199	HEADREST LOCKING COLLAR	\$12.00
3	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$2.00
4	1		1005791	MULTIAXIAL HEADREST CURVED VERT POST WELDMENT	\$52.00
	1		1007005	MULTIAXIAL HEADREST CURVED VERT POST + 4" WELDMENT	\$72.00
	1		1020201	MULTIAXIAL HEADREST STRAIGHT VERT POST WELDMENT	\$47.00
5	1		1007001	BALL-JOINT, 1" DIA CURVED	\$27.00
6	1		1015360	1/4-20 RATCHET HANDLE	\$17.00
7	1		1016938	SINGLE HANDLE MAIN HOUSING RT	\$57.00
8	1		1016939	SINGLE HANDLE MAIN HOUSING LT	\$57.00
	1		1016937	KIT, INFINITE CURVED POST STANDARD (ITEMS 1-8)	\$215.00
	1		1021428	KIT, INFINITE CURVED POST + 4" (ITEMS 1-8)	\$235.00
	1		1020999	KIT, INFINITE STRAIGHT POST (ITEMS 1-8)	\$210.00


NOTES:

HEADREST HORIZONTAL TUBES


HEADREST FIXED HORIZONTAL TUBE

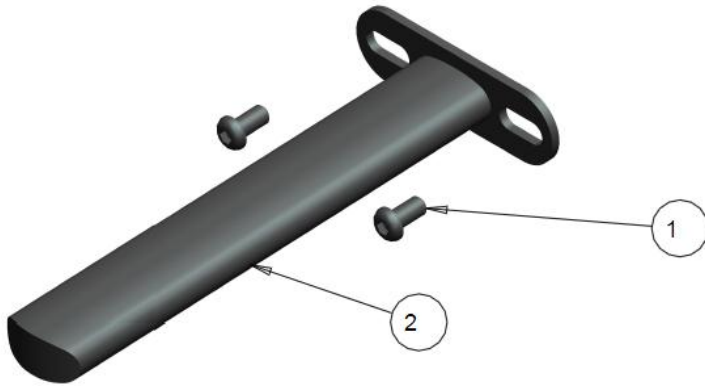

HEADREST ARTICULATED HORIZONTAL TUBE

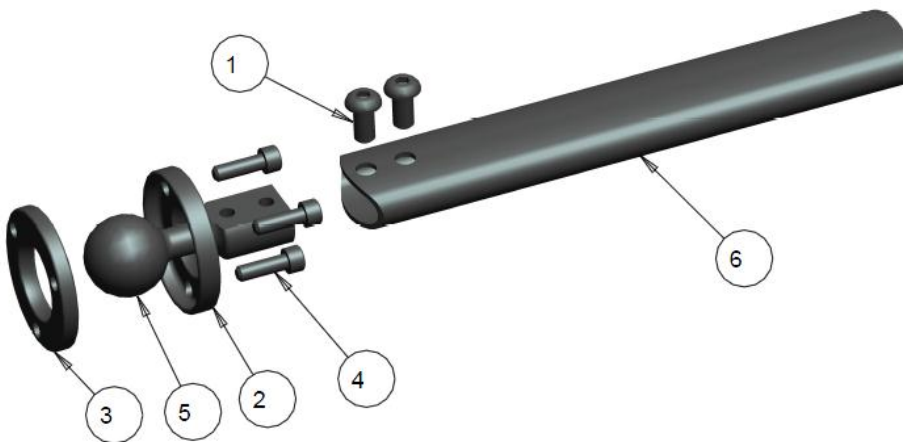

HEADREST INFINITE HORIZONTAL TUBE

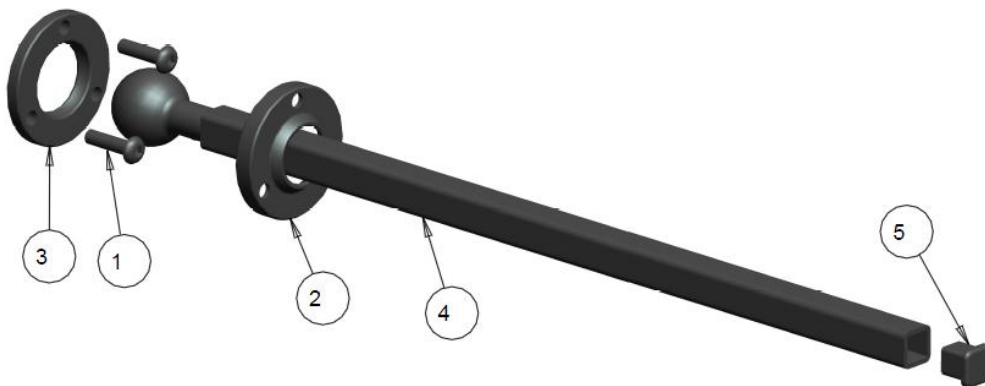

HEADREST FIXED HORIZONTAL TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1000790	HORIZONTAL FIXED HEADREST WELDMENT	\$49.00
	1		1003047	HORIZONTAL FIXED HEADREST WELDMENT +4"	\$59.00
	1		1001208	KIT, HORIZONTAL FIXED HEADREST (ITEMS 1-2)	\$52.00
	1		1004365	KIT, HORIZONTAL FIXED HEADREST +4" (ITEMS 1-2)	\$62.00

NOTES:

HEADREST ARTICULATED HORIZONTAL TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001198	FACE PLATE, BALL SOCKET CLAMP	\$14.00
3	1		1001200	SPACER PLATE, BALL SOCKET CLAMP	\$12.00
4	3		1001203	SCREW, SOCKET HEAD CAP, 10-32NF X 5/8	\$3.00
5	1		1002533	BALL-JOINT, 1" DIA	\$25.00
6	1		1001210	HEADREST HORIZONTAL ARTICULATED WELDMENT	\$40.00
	1		1003980	HEADREST HORIZONTAL ARTICULATED +4" WELDMENT	\$50.00
	1		1003427	KIT, HORIZONTAL ARTICULATED HEADREST (ITEMS 1-6)	\$90.00
	1		1003996	KIT, HORIZONTAL ARTICULATED HEADREST +4" (ITEMS 1-6)	\$100.00

NOTES:


HEADREST INFINITE HORIZONTAL TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	3		1000937	SCREW, SOCKET HEAD BUTTON, #10-32NF X 3/4	\$3.00
2	1		1001198	FACE PLATE, BALL SOCKET CLAMP	\$14.00
3	1		1001200	SPACER PLATE, BALL SOCKET CLAMP	\$12.00
4	1		1021420	HORIZONTAL SQ TUBE WELDMENT	\$63.00
	1		1016941	HORIZONTAL SQ TUBE WELDMENT (+4")	\$68.00
5	1		1016945	SQUARE ENDPLUG 0.5 X 0.5	\$2.50
	1	A	1021429	KIT, HORIZONTAL INFINITE HEADREST (ITEMS 1-5)	\$89.00
	1	A	1016943	KIT, HORIZONTAL INFINITE HEADREST +4" (ITEMS 1-5)	\$94.00

NOTES: A ONLY TO BE USED WITH INFINITE VERTICLE TUBE HARDWARE

LINKS HEADREST

LINKS HEADREST STRAIGHT VERTICLE TUBE

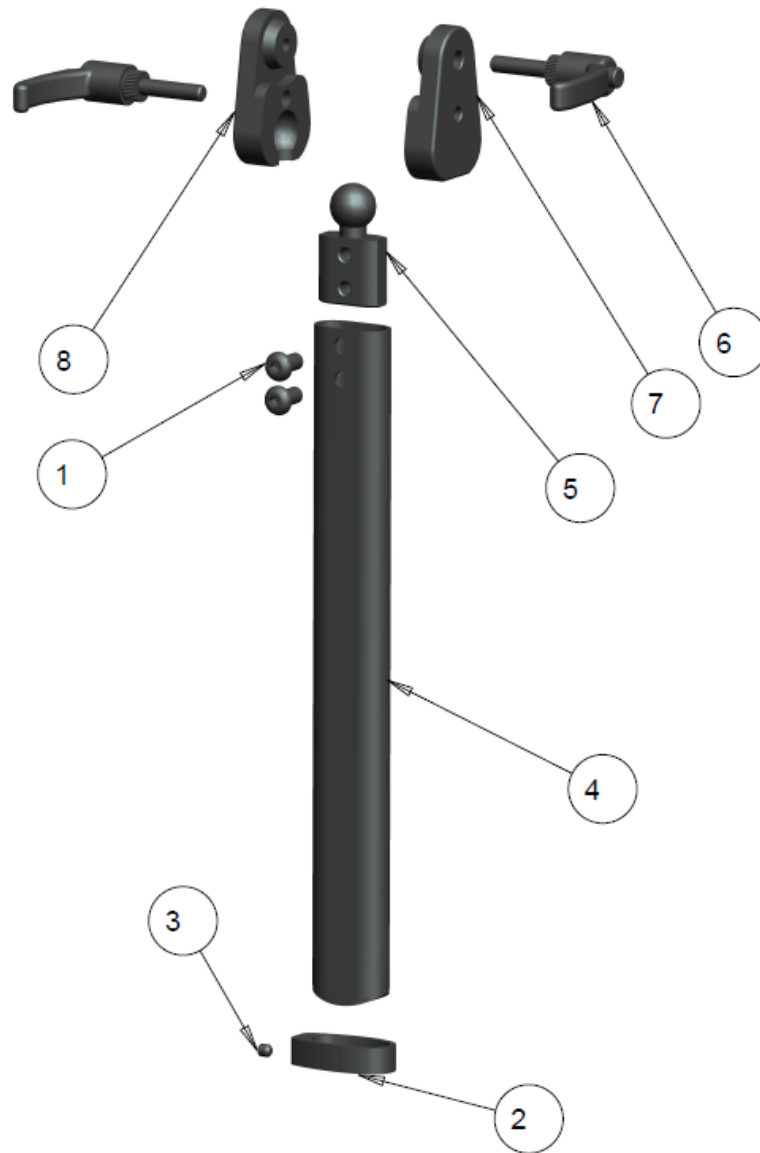

LINKS HEADREST STRAIGHT VERTICLE TUBE TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1001199	HEADREST LOCKING COLLAR	\$12.00
3	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$2.00
4	1		1014048	VERTICLE OVAL TUBE LINKS WELD	\$44.00
5	1		1014268	BALL-JOINT, .687 DIA	\$32.00
6	2		1015360	1/4-20 RATCHET HANDLE	\$17.00
7	1		1015362	BALL JOINT HANDLE MAIN HOUSING RT	\$58.00
8	1		1015363	BALL JOINT HANDLE MAIN HOUSING LT	\$58.00
	1		1015361	KIT, LINKS STRAIGHT POST (ITEMS 1-8)	\$196.00

NOTES:

LINKS HEADREST CURVED VERTICLE TUBE




LINKS HEADREST CURVED VERTICLE TUBE TABLE


ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	1		1001199	HEADREST LOCKING COLLAR	\$12.00
2	1		1001214	SCREW, SET, SOC HD CUP POINT #10-32 X 3/16	\$2.00
3	2		1001381	SOC HD SCREW 1/4-20 X 1.00	\$3.00
4	2		1015360	1/4-20 RATCHET HANDLE	\$17.00
5	1		1015362	BALL JOINT HANDLE MAIN HOUSING RT	\$58.00
6	1		1015363	BALL JOINT HANDLE MAIN HOUSING LT	\$58.00
7	1		1015445	CURVED VERTICLE OVAL TUBE LINKS	\$63.00
8	1		1015448	BALL-JOINT, .687 DIA (NOT THREADED)	\$32.00
	1		1015444	KIT, LINKS CURVED POST (ITEMS 1-8)	\$206.00

NOTES:

LINKS HEADREST BALL JOINT

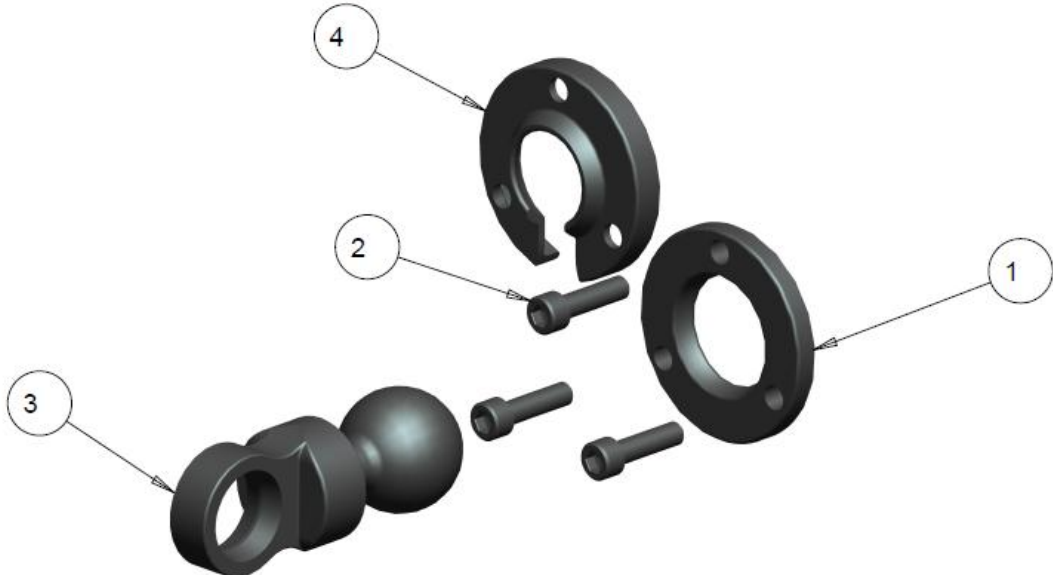

LINKS HEADREST SINGLE LINK

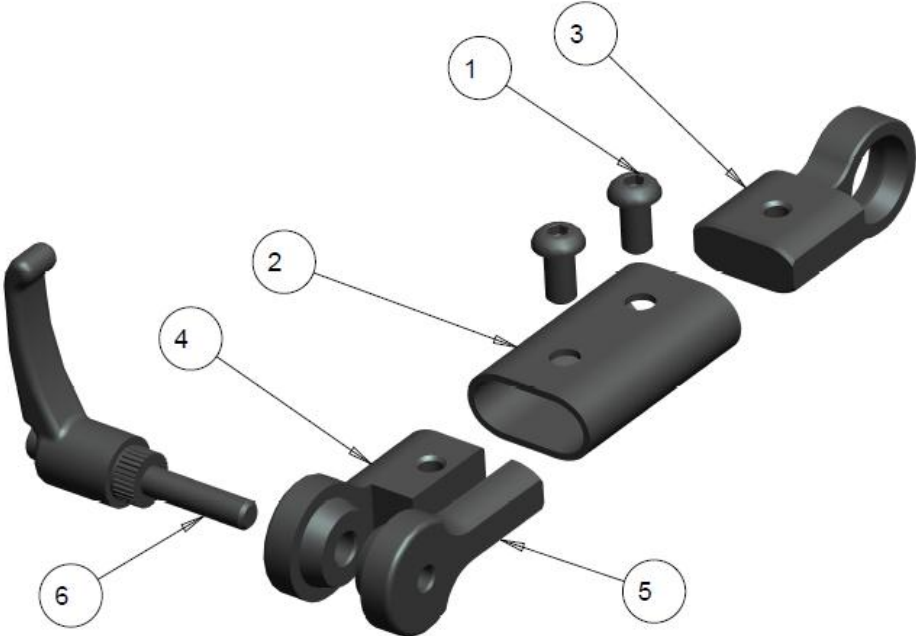

LINKS HEADREST BALL JOINT TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	1		1001200	SPACER PLATE, BALL SOCKET CLAMP	\$12.00
2	3		1006241	SCREW, SOCKET HEAD CAP, 10-32NFX 3/4	\$3.00
3	1		1014042	COBRA BALL JOINT MACHINED	\$25.00
4	1		1014045	SLOTTED INSIDE PLATE- BALL JOINT	\$14.00
	1		1014044	KIT, LINKS BALL JOINT (ITEMS 1-4)	\$44.00

NOTES:


LINKS HEADREST SINGLE LINK TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	2		1000411	SCREW, SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
2	1		1014032	MAIN OVAL TUBE HOUSING	\$16.00
3	1		1014033	RING STRAIGHT LINK MACHINED	\$18.00
4	1		1014034	CONE STRAIGHT LINK MAIN	\$14.00
5	1		1014035	CONE STRAIGHT LINK INSERT	\$10.00
6	1		1015360	1/4-20 RATCHET HANDLE	\$17.00
	1		1014031	KIT, ADDITIONAL LINK SINGLE (ITEMS 1-6)	\$50.00

NOTES:

HEADREST BASE ADAPTOR BRACKET

2-HOLE HEADREST BASE ADAPTOR BRACKET

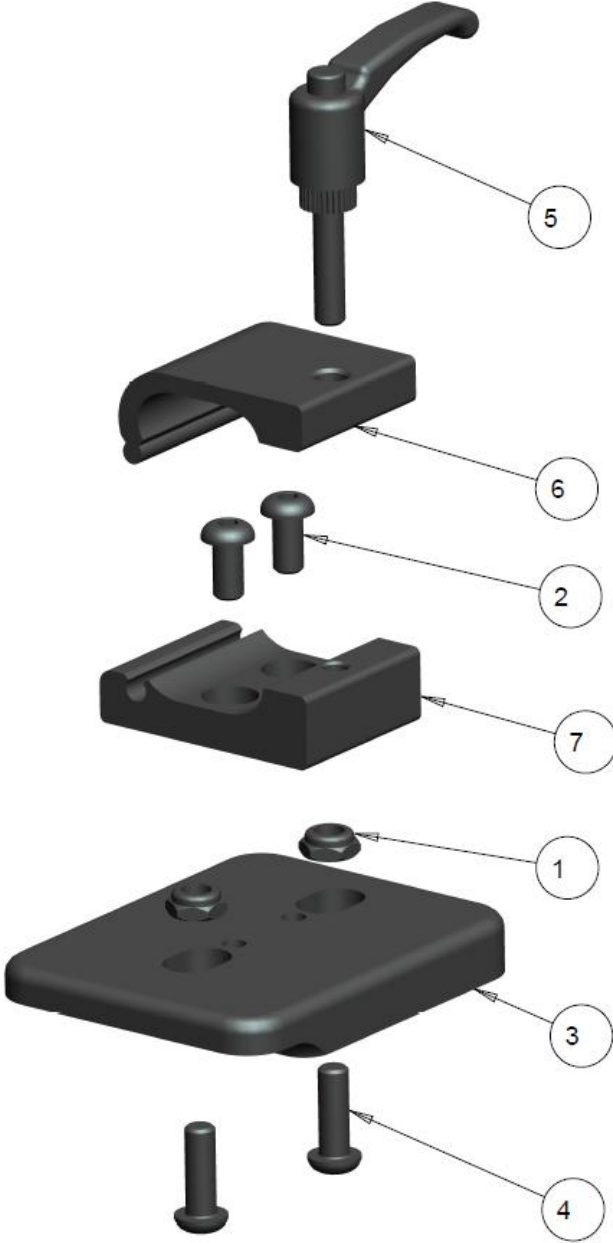


2-HOLE HEADREST BASE ADAPTOR BRACKET TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	2		1000397	NUT, HEX-NYLOCK THIN, 1/4-20NC	\$2.00
2	2		1000411	SCREW, SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
3	1		1001367	2-HOLE MOUNTING PLATE	\$50.00
4	2		1001391	SCREW, SOCKET HEAD BUTTON, 1/4-20NC X 3/4	\$3.00
5	1		1015360	1/4-20 RATCHET HANDLE	\$17.00
6	1		1019474	OVAL UPPER CLAMP	\$16.00
7	1		1019475	OVAL TUBE LOWER CLAMP	\$20.00
	1		1001209	KIT, 2-HOLE HEADREST ADAPTOR BRACKET (ITEMS 1-7)	\$105.00

NOTES:

UNIVERSAL HEADREST BASE ADAPTOR BRACKET

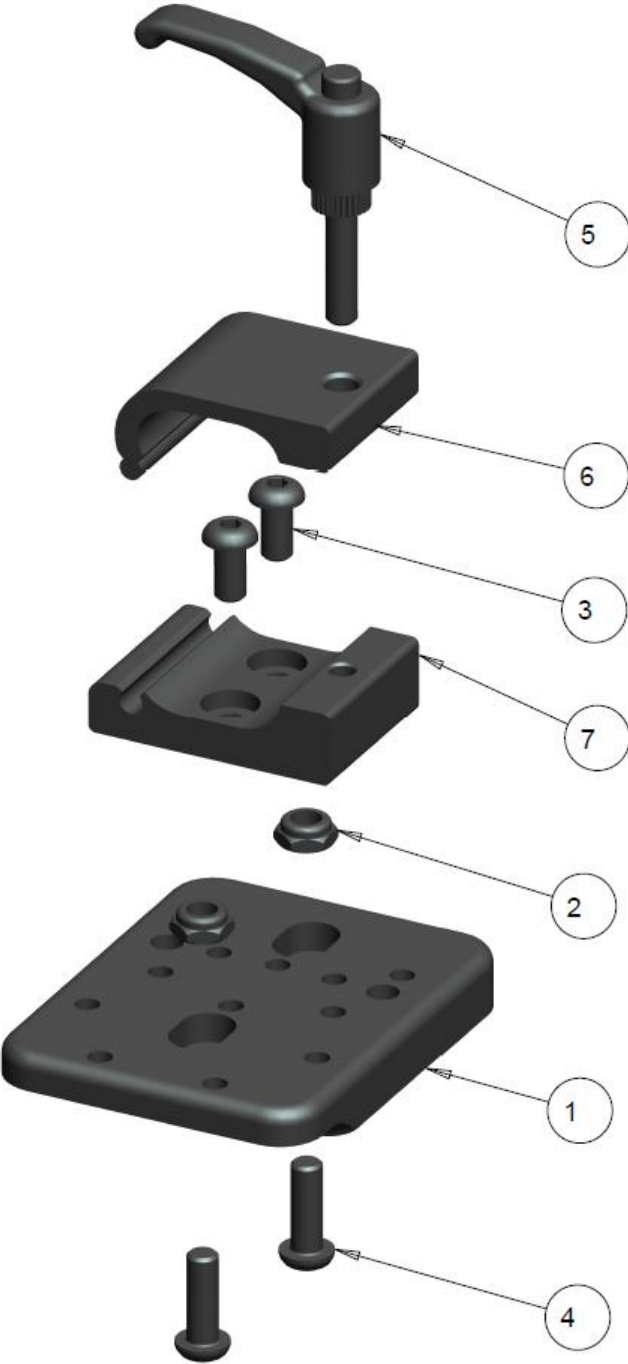

UNIVERSAL HEADREST BASE ADAPTOR BRACKET TABLE

ITEM	QTY	NOTES	PART NO.	DESCRIPTION	PRICE/ UNIT
1	1		1000218	UNIVERSAL MOUNTING PLATE, HEADREST	\$61.00
2	2		1000397	NUT, HEX-NYLOCK THIN, 1/4-20NC	\$2.00
3	2		1000411	SCREW,SOC BUTTON HEAD 1/4-20 X 0.500	\$3.00
4	2		1001391	SCREW, SOCKET HEAD BUTTON, 1/4-20NC X 3/4	\$3.00
5	1		1015360	1/4-20 RATCHET HANDLE	\$17.00
6	1		1019474	OVAL UPPER CLAMP	\$16.00
7	1		1019475	OVAL TUBE LOWER CLAMP	\$20.00
	1		1003995	KIT, UNIVERSAL HEADREST ADAPTOR BRACKET (ITEMS 1-7)	\$120.00

NOTES: