**FUTURE
MOBILITY**

SEATING & POSITIONING
SOLUTION
Manufacturer: Future Mobility

USER MANUAL

Overbed Table - Table de lit

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SPECS

BASE	19.5"W x 28"L
COLOUR	Base is white, support column is powder-coated, table top is walnut wood-grain
HEIGHT ADJUSTMENT	Approx. 30" - 45"
PRODUCT WEIGHT	30 lbs
PRODUCT WEIGHT CAPACITY	40 lbs.
WARRANTY	1 year limited

FEATURES

- Light upward touch required for lever and activating the spring-loaded mechanism allowing the table to elevate freely
- Unique H-base design permits use with chairs
- Attractive walnut woodgrained laminate top
- Durable, heavy duty bases, painted welded tubular steel construction
- Supports up to 40 pounds of evenly distributed weight

CANADA

6750 Professional Court,
Mississauga, Ontario, L4V 1X6
1-888-737-4011

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TOOLS REQUIRED: 7/16 and 9/16 Wrench, 5/32 Allen key
OUTILS REQUIS: 7/16 et 9/16 Clé, 5/32 Clé Allen

ATTACH BASE TO POST - FIXEZ LA BASE AU POTEAU

1

Insert the bolts and plate combination through the base to attach post.
Use 9/16 wrench to tighten.

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1

Slide the table top into the post (1).
Tighten with 7/16 wrench.
Reassemble bolt, nut and ratchet lever to post (2). Remove safety bolt (3).

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3

OPERATING INSTRUCTIONS - INSTRUCTIONS D'UTILISATION

(1) Shows the overbed table fully assembled. The table height can be adjusted by squeezing the ratchet lever towards the table top (2). Using your other hand support the other end of the table top while adjusting the height.

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